
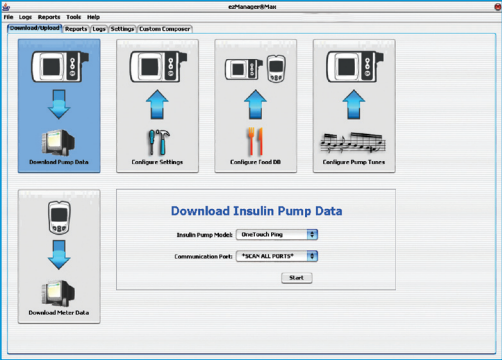
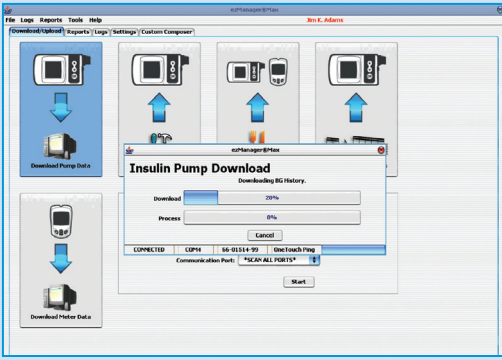
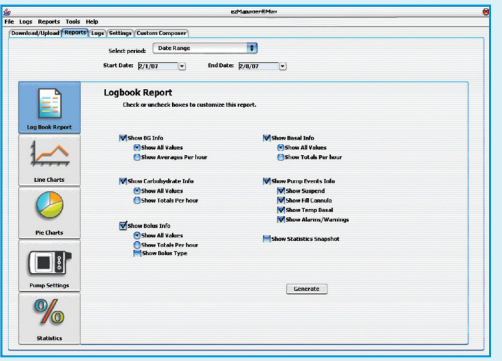



## Downloading an Animas<sup>®</sup> 2020 or OneTouch<sup>®</sup> Ping<sup>™</sup> Insulin Pump\* to ezManager<sup>®</sup> Max

<p><b>1</b> Prepare the pump and infrared dongle (wireless interface). Have patient disconnect and suspend pump. Remove belt clip from pump.</p>	<p><b>Pump:</b></p> <ol style="list-style-type: none"> <li>1) From the MAIN MENU, scroll to "Suspd/Resum" and press <b>OK</b>.</li> <li>2) "Suspend" is highlighted. Press <b>OK</b>.</li> </ol> <p><b>Infrared Dongle:</b></p> <ol style="list-style-type: none"> <li>1) Ensure the dongle is plugged into the computer.</li> <li>2) Place the head of the dongle on a flat surface.</li> <li>3) Lay the pump on the dongle so that it is positioned as seen in Figure 1.</li> </ol>	 <p>Figure 1</p>
<p><b>2</b> Prepare the software for download.</p>	<p><b>Software:</b></p> <ol style="list-style-type: none"> <li>1) Login to the program and open the desired User Account (or click "Add Account" for a new patient).</li> <li>2) On the "Download/Upload" sheet tab, click the button for "Download Pump Data" (Refer to Figure 2).</li> <li>3) In the lower, right-hand corner, select the insulin pump model.</li> <li>4) Click the "Start" button.</li> <li>5) Click "OK" when reminded to disconnect and suspend the pump.</li> <li>6) On the pump, press <b>OK</b> to ensure the screen is lit until a connection is established.</li> </ol>	 <p>Figure 2</p>
<p><b>3</b> Ensure that a connection is established.</p>	<p><b>Connection:</b></p> <ol style="list-style-type: none"> <li>1) The lower, left-hand corner of the download box will read "Searching" and will change to "Connected" when a connection is established (Refer to Figure 3).</li> <li>2) The pump type and serial number will also appear when the connection is established.</li> <li>3) Click "OK" when informed "Download Completed."</li> </ol>	 <p>Figure 3</p>

## Generating and Printing an ezManager<sup>®</sup> Max Report or Log

<p><b>4</b> Generate</p>	<ol style="list-style-type: none"> <li>1) Click the Reports or Logs tab.</li> <li>2) Select the desired Date Range.</li> <li>3) Click the desired Report or Log button (e.g., Logbook, Refer to Figure 4).</li> <li>4) Once you are satisfied with your settings, click "Generate."</li> <li>5) Repeat steps 1 through 4 as necessary for additional Reports or Logs.</li> </ol> <p><b>NOTE:</b> Download information is automatically saved to the program.</p>	 <p>Figure 4</p>
<p><b>5</b> Print</p>	<ol style="list-style-type: none"> <li>1) After generating the report, click the Printer icon at the top left of the screen (Refer to Figure 5).</li> <li>2) When the print window appears, click "OK" if the proper printer and settings have been selected.</li> </ol>	 <p>Figure 5</p>

Software Technical Support

1-877-937-7867  
Monday - Friday, 8:00AM - 5:00PM (Eastern Time)

ezmanager@animus.jnj.com

**NOTE:** This reference guide is not intended as a substitute for the ezManager<sup>®</sup> Max Diabetes Management Software User Guide. Please refer to the User Guide for full product information.

\*Limited downloads for Animas<sup>®</sup> IR 1000, IR 1200, and IR 1250 insulin pumps.

P/N: 420-390-00 Rev A © 2009 Animas Corporation 2009/04 Rx Only. AN09-098