

**ANEMIA METABOLITE LABORATORY
AURORA, COLORADO**

Specimen Requirements:

1. Serum, urine, Heparin Plasma and CSF are acceptable samples (2-methylcitric acid 1 & 2 cannot be done on citrated plasma).
2. Sample volume is 1 ml. The minimum sample volume is 300 µl.
3. Serum and plasma should be separated from cells within 1 hour from time of collection or kept on ice prior to separation.
4. Samples can be stored frozen indefinitely, refrigerated for several weeks and at room temperature up to 24 hours.
5. Sample date, sample ID number, sample type are required on the paperwork sent with the samples.
6. Phone number and FAX number, name, position and address of person to whom the results should be sent.

Unacceptable Samples:

1. Unidentified or undated samples, samples <300 µl in volume, whole blood or samples obviously contaminated with bacteria.

Shipping:

Samples should be shipped frozen on dry ice in freezer-safe containers to prevent spillage.

Federal Express priority overnight shipment is preferred for delivery by 10:30am on Monday through Friday. Saturday deliveries are not accepted and we are closed on most major holidays.

Shipping address to be used:

*Sally P. Stabler, M.D.
Anemia Metabolite Laboratory
12700 E. 19th Avenue, RC2, Room 9490E
MS B170
Aurora, CO 80045*

Technicians:

Whitney Phinney, PRA

Phone: (303)-724-4086 – or – (303)-724-4092

FAX: (303)-724-4087

CPT Codes: MMA 83921 (x 2)
HCYS 82136

CLIA License #06D0644360, Expires 08/2021